

OPERATING INSTRUCTIONS

Soltice® II Recliner

September 2017

Overview

KI's Soltice II Recliner is a responsive healthcare solution that is durable, intuitive and elegant. Soltice II has ample proportions and sturdy construction to accommodate all body types up to 750 lbs. Its recline-release feature is fingertip activated, making it easy to use. The ottoman is lever actuated and operates independently of the backrest. The central locking caster mechanism allows the casters to be locked or unlocked from one location.

Central Locking Caster Mechanism

The Central Locking Caster Mechanism has three positions that are activated by stepping down on the corresponding color on the mechanism lever.

- The “neutral” position (pictured), allows the chair to rotate 360 degrees, allowing the chair to maneuver in tight spaces.
- The “green” position locks-out one of the front casters allowing the chair to track straight while in motion.
- The “red” position locks 3 out of 4 casters keeping the chair stationary and should be used for patient ingress/egress from the chair.



Central locking 5" sealed caster

Backrest Operation

To recline the backrest, pull out and hold the backrest release lever while applying pressure against the backrest of the chair. The backrest is infinitely adjustable to 175 degrees. When the desired position is achieved, let go of the backrest release lever and the chair will maintain that position. To return to the upright position, pull out and hold the backrest release lever while sitting upright.



Independent backrest actuator

Ottoman Operation

The ottoman operates independent of the backrest and is engaged by pulling back on the ottoman actuator handle.



Independent footrest actuator

Pull-out Footrest (Transporter Base)

If needed, there is a pull out footrest on casters located under the chair. The footrest can be engaged by simply pulling out on the front of the footrest. The footrest can be in the outward position while the chair is in motion.



Transporter base and pull-out footrest

Key Features & Options

- Seam saver weltcord
- Field-replaceable wood or poly arm caps
- Durable double needle stitching
- Pull-out footrest
- 5" sealed casters
- Ergonomic push bars
- 360 degree turning radius
- Clean-out space between seat and back
- Replaceable seat and back covers
- Central locking casters
- Independent backrest and ottoman activation
- Infinite adjustment on backrest
- Optional tablet arms/tables, magazine pouch, catheter hook, and IV pole

